## PRO SALUTATE ANIMALIS

### ZAHNREINIGER DENTAL MICRO-FLEECE

D

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### DENTAL MICRO-FLEECE TOOTH CLEANING

GB

The ST. DIEM'S DENTAL MICRO-FLEECE effectively cleans teeth, gums and tongue of dogs and cats. The soft microfibre material removes deposits and plaque. Thanks to its silver ion technology, the DENTAL MICRO-FLEECE binds bacteria. Actively combats tartar and plaque. Reduces halitosis.

#### How to use:

Slip your index finger into the DENTAL MICRO-FLEECE. Slightly moisten the tip of the DENTAL MICRO-FLEECE. Clean all sides of each tooth with light circular movements. Following this, you can then massage the gums and clean the tongue. After each use, rinse with warm water and allow to dry.

For a better result apply ST. DIEM'S TOOTHPASTE onto the DENTAL MICRO-FLEECE.

Disinfection (by boiling or microwave/steam sterilisation) is possible.

#### Notes:

- Replace the DENTAL MICRO-FLEECE if damaged or the tip turns yellow.
- When used regularly, replace every 4-5 weeks.
- The DENTAL MICRO-FLEECE is not suitable for washing machines, dishwashers, chemical cleaning or cold water sterilisation. Do not bleach.
- The microfibre used corresponds to the OekoTex 100 standard.
- Do not allow the dog or cat to swallow the DENTAL MICRO-FLEECE.
- Rinse with warm water before using for the first time.

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FINIDORE Manufaktur GmbH