D ZAHNREINIGER DENTAL MICRO-FLEECE

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G DENTAL MICRO-FLEECE TOOTH CLEANING

The ST. DIEM'S DENTAL MICRO-FLEECE effectively cleans teeth, gums and tongue of dogs and cats. The soft microfibre material removes deposits and plaque. Thanks to its silver ion technology, the DENTAL MICRO-FLEECE binds bacteria. Actively combats tartar and plaque. Reduces halitosis.

How to use: Slip your index finger into the DENTAL MICRO-FLEECE. Slightly moisten the tip of the DENTAL MICRO-FLEECE. Clean all sides of each tooth with light circular movements. Following this, you can then massage the gums and clean the tongue. After each use, rinse with warm water and allow to dry. For a better result apply ST. DIEM'S TOOTH PASTE onto the DENTAL MICRO-FLEECE. Desinfection (by boiling or steam sterilisation) is possible. Please note that the finger cot shrinks and the colors may fade.

Notes:

- Replace the DENTAL MICRO-FLEECE if damaged or the tip turns yellow.
- When used regularly, replace every 4-5 weeks.
- The DENTAL MICRO-FLEECE is not suitable for washing machines, dishwashers, microwave sterilisation, chemical cleaning or cold water sterilisation. Do not bleach.
- The microfibre used corresponds to the Standard 100 by OEKO-TEX®.
- Do not allow the dog or cat to swallow the DENTAL MICRO-FLEECE.
- Rinse with warm water before using for the first time.

Fibre/Yarn contains a Biocidal Product:

Contains „Silver Sodium Zirconium Hydrogenphosphate - CAS-# 265647-11-8“ built into the fiber polymer to counter the settlement and growth of micro-organisms and the development of odor in/on the treated article.

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FINIDORE Manufaktur GmbH, Kirchstraße 3, D-86926 Beuern am Ammersee